

PRODUCT INFORMATION

ISYSOFT Cleaning wipes

Area of application and properties

- + **Intended use:** Wipes for cleaning, cleaning, especially suitable for cleaning surfaces contaminated with CMR¹⁾ drugs
- + **Properties:** Extremely absorbent cloths, folded several times, suitable for the daily cleaning routine and for absorbing spilled liquids. The cloths can be used dry or moistened/wet. ISYSOFT-cleaning cloths are low in lint and are highly effective and are highly effective at cleaning. For single use only!
- + **Recommended use:** For the cleaning of cytostatic contaminations we recommend, based on a Study by the German Institute for Energy and Environmental Technology (IUTA e.V.):
Cleaning in 2 steps:
first using a) an alkaline cleaning agent > pH 11 and b) Isopropanol (70%)
In a study on the subject of cleaning cloths carried out by the Institute for Energy and Environmental Technology, highly contaminated surfaces (with different cytostatics) were cleaned using ISYSOFT clean-ing cloths, which reached a degree of cleanliness of 99.95%. Wipes are moistened with the cleaning solution, avoiding spray. In case of highly contaminated surfaces (e.g. after spills, etc. the 2-step process should be repeated 3 times.
- + **Disposal:** If cloths have been used to clean contaminated areas (CMR-drugs): waste requiring supervision (waste code: 18 01 04 in accordance with 2000/532/EC); in case of high contamination, waste requiring special supervision (waste code: 18 01 08* in accordance with 2000/532/EC); collect and dispose of waste separately.
¹⁾: Carcinogenic, mutagenic, reprotoxic

Versions ISYSOFT

Size	400 x 300 mm
Order No. non -sterile	121500
Order No. sterile	121501
Colour	Natural white
Packaging unit non-sterile	1 PU = 100 pieces (4 x 25 wipes)
Packaging unit sterile	1 PU = 100 pieces

Material properties ISYSOFT

Material	Low particle airlaid paper
Weight	60 g/m ²
Absorbable liquid volume	≤ 65 ml

¹⁾: Internal test procedure.

Versions ISYSOFT II (latex-free)

Size	380 x 300 mm
Order No. non-sterile	121507
Order No. sterile	121508
Colour	white
Packaging unit non-sterile	1 PU = 100 pieces (4 x 25 cloths)

Packaging unit sterile

1 PU = 100 pieces
(50 x 2 cloths, sterile wrapped - optimal for 2-step cleaning)

Material properties ISYSOFT II (latex-free)

Material	Viscose/polyester-mix, latex-free
Weight	65g/m ²
Cleanroom suitability	ISO class 5
Absorbable liquid volume	≤ 70 ml

Sterilisation ISYSOFT I and ISYSOFT II (latex-free)

Process	Fumigation with ethylenoxide
----------------	------------------------------

Quality management system

Our **quality management system** is **tested and certified** by TÜV Product Service GmbH (a certification body accredited by the German Accreditation Council,) in accordance with DIN EN ISO 9001:2015. Regular **audits and production site inspections** guarantee the quality of our products.

Storage and transport requirements

- + Dark (protect from direct UV light and sunlight)
 - + Cool (-5 to +40°C)
 - + Dry (relative humidity 30%- 60%)
 - + No contact with pointed and/or sharp objects
-

Shelf life

- + 3 years from the date of manufacture
-

Barcode-system für die Lagerhaltung

- + On all packaging units
 - + Item-nos. are readable as EAN-13 code
 - + Manufacturing and use-by date as well as lot number (LOT) is readable as code 39
-

Manufacturer / Distributor

Berner International GmbH, Werner-von-Siemens-Str. 19, 25337 Elmshorn, Germany
Tel: +49 4121 43560, Fax: +49 4121 435620
info@berner-safety.de, www.berner-safety.de